



Eyezen™ +

Fitting Eyezen™ + Lenses

Eyezen+ lenses require the following standard measurements:

- Monocular Distance PD
- Fitting Height: Simply measure from center pupil to the lowest point of the lens to acquire fitting height. For **Eyezen+**, minimum fitting height is **15mm**.

Eyezen+ lenses with embedded Harmful Blue Light* protection are also recommended with Crizal® No-Glare treatment for our most comprehensive eye protection.

The following laser engravings located on the nasal side of each lens will identify the **Eyezen** design received.

Eyezen+ 0



Eyezen+ 1



Eyezen+ 2



Eyezen+ 3



Eyezen+ 4



Embedded **Harmful Blue Light*** Protection.

* *Eyezen* lenses block at least 20% of Harmful Blue Light, which is the blue-violet wavelengths between 415-455nm on the light spectrum believed most toxic to retinal cells.

EYEZEN™+ LENSES AVAILABILITY

for Eyecare Professionals

Effective **DECEMBER 2019**

MATERIAL	LIGHT MANAGEMENT	INDEX	COLOR	POWER RANGE	BASE CURVE	MAX CYLINDER
PLASTIC	Clear	1.5		-10.00 to +7.50	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
	Essilor Polarized	1.5	Gray, Brown, Gray-Green, Gray-1, Brown-1	-10.00 to +7.50	2.00, 3.25, 4.25, 6.25, 7.25, 8.25	-6.00
	Xperio UV™ Polarized	1.5	Gray, Brown, Gray-Green, Gray-1, Brown-1	-10.00 to +7.50	2.00, 3.25, 4.25, 6.25, 7.25, 8.25	-6.00
	Transitions® Signature® GEN 8™	1.5	Gray, Brown, Graphite Green	-10.00 to +7.50	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
	Transitions® Signature® Style Colors	1.5	Amber, Amethyst, Emerald, Sapphire	-10.00 to +7.50	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
	Transitions® Vantage®	1.5	Gray	-10.00 to +7.50	2.25, 4.25, 5.25, 8.25	-6.00
	Transitions® XTRActive® Gray*	1.5	Gray	-10.00 to +7.50	2.25, 4.25, 5.25, 6.25, 8.25	-6.00
	Transitions® XTRActive® Brown	1.5	Brown	-10.00 to +7.50	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
TREXA™	Transitions® XTRActive® Style Mirrors	1.5	Blue, Green, Gold, Pink, Red, Silver Shadow	-10.00 to +7.50	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
	Clear §	1.53		-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
AIRWEAR®	Transitions® Signature® GEN 8™ §	1.53	Gray, Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25, 8.75	-6.00
	Clear	1.59		-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Essilor Polarized	1.59	Gray, Brown, Gray-Green, Gray-1, Brown-1, Copper, Ruby	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00/7.50, 8.50	-6.00
	Xperio UV™ Polarized	1.59	Gray, Brown, Gray-Green, Gray-1, Brown-1, Copper, Ruby	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00/7.50, 8.50	-6.00
	Transitions® Signature® GEN 8™	1.59	Gray, Brown, Graphite Green	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® Signature® Style Colors	1.59	Amber, Amethyst, Emerald, Sapphire	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® Vantage®*	1.59	Gray	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® XTRActive®	1.59	Gray, Brown	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
THIN & LITE®	Transitions® XTRActive® Style Mirrors	1.59	Blue, Green, Gold, Pink, Red, Silver Shadow	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Colors™*	1.59		-10.00 to +8.00	2.25, 4.25, 6.50, 7.50, 8.50	-6.00
	Clear	1.6		-12.00 to +8.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.50	-6.00
	Transitions® Signature® GEN 8™	1.6	Gray, Brown	-12.00 to +8.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.50	-6.00
	Clear	1.67		-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	Essilor Polarized	1.67	Gray, Brown, Gray-Green	-14.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-4.00
	Xperio UV™ Polarized	1.67	Gray, Brown, Gray-Green	-14.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-4.00
	Transitions® Signature® GEN 8™	1.67	Gray, Brown, Graphite Green	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	Transitions® Signature® Style Colors	1.67	Amber, Amethyst, Emerald, Sapphire	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	Transitions® XTRActive®	1.67	Gray, Brow	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	Transitions® XTRActive® Style Mirrors	1.67	Blue, Green, Gold, Pink, Red, Silver Shadow	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	Clear	1.74		-20.00 to +13.00	0.75,1.75,2.75,3.75,5.25,6.50,8.00,10.00	-6.00
	Transitions® Signature® GEN 8™	1.74	Gray	-20.00 to +13.00	0.75,1.75,2.75,3.75,5.25,6.50,8.00,10.00	-6.00

* Transitions Vantage, AIRWEAR® Colors™, 1.50 plastic Transitions® XTRActive® Gray available on Eyezen+ 0-3 only

EYEZEN LENS DESIGNS

Eyezen+ 0	patients age 17 and under
Eyezen+ 1	patients age 18 to 34
Eyezen+ 2	patients age 35 to 39
Eyezen+ 3	patients age 40 to 44
Eyezen+ 4	patients age 45 to 50 ¹

There are five unique **Eyezen+** designs. Although the recommended age ranges for each design provided below are based on research, as always, the Eyecare Professional is free to use discretion in prescribing **Eyezen+** enhanced single vision lenses.

¹ Eyezen+ lenses are not adapted to patients that have been prescribed progressive lenses.



EyezenTM START

Fitting EyezenTM Lenses

Eyezen START lenses require the following standard measurements:

- Monocular Distance PD
- Fitting Height: Simply measure from center pupil to the lowest point of the lens to acquire fitting height.
There is no minimum fitting height

Eyezen START lenses with embedded Harmful Blue Light* protection are always recommended with Crizal[®] No-Glare treatment for our most comprehensive eye protection.

The following laser engraving, located on the nasal side of each lens, will identify the **Eyezen START** design received.

Eyezen Start



Embedded **Harmful Blue Light* Protection.**

* Eyezen START lenses block at least 20% of Harmful Blue Light, which is the blue-violet wavelengths between 415-455nm on the light spectrum believed most toxic to retinal cells.

EYEZEN™ START LENSES AVAILABILITY

for Eyecare Professionals

Effective **DEC 2019**

MATERIAL	LIGHT MANAGEMENT	INDEX	COLOR	POWER RANGE	BASE CURVE	MAX CYLINDER
PLASTIC	Clear	1.5		-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
	Essilor Polarized §	1.5	Gray, Brown, Gray-Green, Gray-1, Brown-1	-10.00 to +6.00	2.00, 3.25, 4.25, 6.25, 7.25	-4.00
	Xperio UV™ Polarized §	1.5	Gray, Brown, Gray-Green, Gray-1, Brown-1	-10.00 to +6.00	2.00, 3.25, 4.25, 6.25, 7.25	-4.00
	Transitions® Signature® GEN 8™	1.5	Gray, Brown, Graphite Green	-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
	Transitions® Signature® GEN 8™ lenses style colors §	1.5	Amber, Amethyst, Emerald, Sapphire	-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
	Transitions® XTRActive® §	1.5	Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
	Transitions® XTRActive® Style Mirrors §	1.5	Blue, Green, Gold, Pink, Red, Silver Shadow	-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
TREXA™	Clear §	1.53		-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
	Transitions® Signature® GEN 8™ §	1.53	Gray, Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 5.25, 6.25, 7.25	-6.00
AIRWEAR®	Clear	1.59		-10.00 to +6.00	1.50, 3.25, 4.25, 4.75, 5.50, 7.00	-6.00
	Essilor Polarized §	1.59	Gray, Brown, Gray-Green, Gray-1, Brown-1, Copper, Ruby	-10.00 to +6.00	1.50, 3.25, 4.25, 5.50, 7.00	-6.00
	Xperio UV™ Polarized §	1.59	Gray, Brown, Gray-Green, Gray-1, Brown-1, Copper, Ruby	-10.00 to +6.00	1.50, 3.25, 4.25, 5.50, 7.00	-6.00
	Transitions® Signature® GEN 8™	1.59	Gray, Brown, Graphite Green	-10.00 to +6.00	1.50, 3.25, 4.25, 4.75, 5.50, 7.00	-6.00
	Transitions® Signature® GEN 8™ lenses style colors §	1.59	Amber, Amethyst, Emerald, Sapphire	-10.00 to +6.00	1.50, 3.25, 4.25, 4.75, 5.50, 7.00	-6.00
	Transitions® XTRActive®	1.59	Gray, Brown	-10.00 to +6.00	1.50, 3.25, 4.25, 4.75, 5.50, 7.00	-6.00
	Transitions® XTRActive® Style Mirrors	1.59	Blue, Green, Gold, Pink, Red, Silver Shadow	-10.00 to +6.00	1.50, 3.25, 4.25, 4.75, 5.50, 7.00	-6.00
THIN & LITE®	Clear	1.6		-12.00 to +8.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.50	-6.00
	Transitions® Signature® GEN 8™	1.6	Gray, Brown	-12.00 to +8.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.50	-6.00
	Clear	1.67		-14.00 to +9.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.00	-6.00
	Essilor Polarized §	1.67	Gray, Brown, Gray-Green	-12.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-4.00
	Xperio UV™ Polarized §	1.67	Gray, Brown, Gray-Green	-12.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-4.00
	Transitions® Signature® GEN 8™	1.67	Gray, Brown	-14.00 to +9.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.00	-6.00
	Transitions® Signature® GEN 8™ lenses style colors §	1.67	Amber, Amethyst, Emerald, Sapphire	-14.00 to +9.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.00	-6.00
	Transitions® XTRActive® §	1.67	Gray, Brown	-14.00 to +9.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.00	-6.00
	Transitions® XTRActive® Style Mirrors §	1.67	Blue, Green, Gold, Pink, Red, Silver Shadow	-14.00 to +9.00	1.75, 2.75, 3.75, 4.75, 6.00, 6.75, 8.00	-6.00
	Clear §	1.74		-20.00 to +13.00	0.75, 1.75, 2.75, 3.75, 5.25, 6.50, 8.00, 10.00	-6.00
Transitions® Signature® GEN 8™ §	1.74	Gray	-20.00 to +13.00	0.75, 1.75, 2.75, 3.75, 5.25, 6.50, 8.00, 10.00	-6.00	

EYEZEN START LENS DESIGNS

LENS ENGRAVING	
PRODUCT	SYMBOL
Eyezen Start	

§ Available on March 10, 2020